



Randy Intervention Truss, Item 1479 Application Instructions

The Randy Intervention Truss (RIT) is a one piece, solid construction, polyethylene plastic device used to prevent SCBA waist straps or other harness type equipment from riding up into the pelvic/thigh gaps of the manikin.

It is constructed in an “H” configuration and the cross bar of the “H” is the part that bridges the groin between the two legs of the Randy manikin you are using. The upright legs of the “H” are the parts that wrap around the thighs of the manikin. These upright portions have holes on each end for the purpose of using the enclosed shoe laces to secure the unit to the legs. The photo below shows its application, but the unit is intended to be used underneath clothing to eliminate any snags that may occur otherwise.

