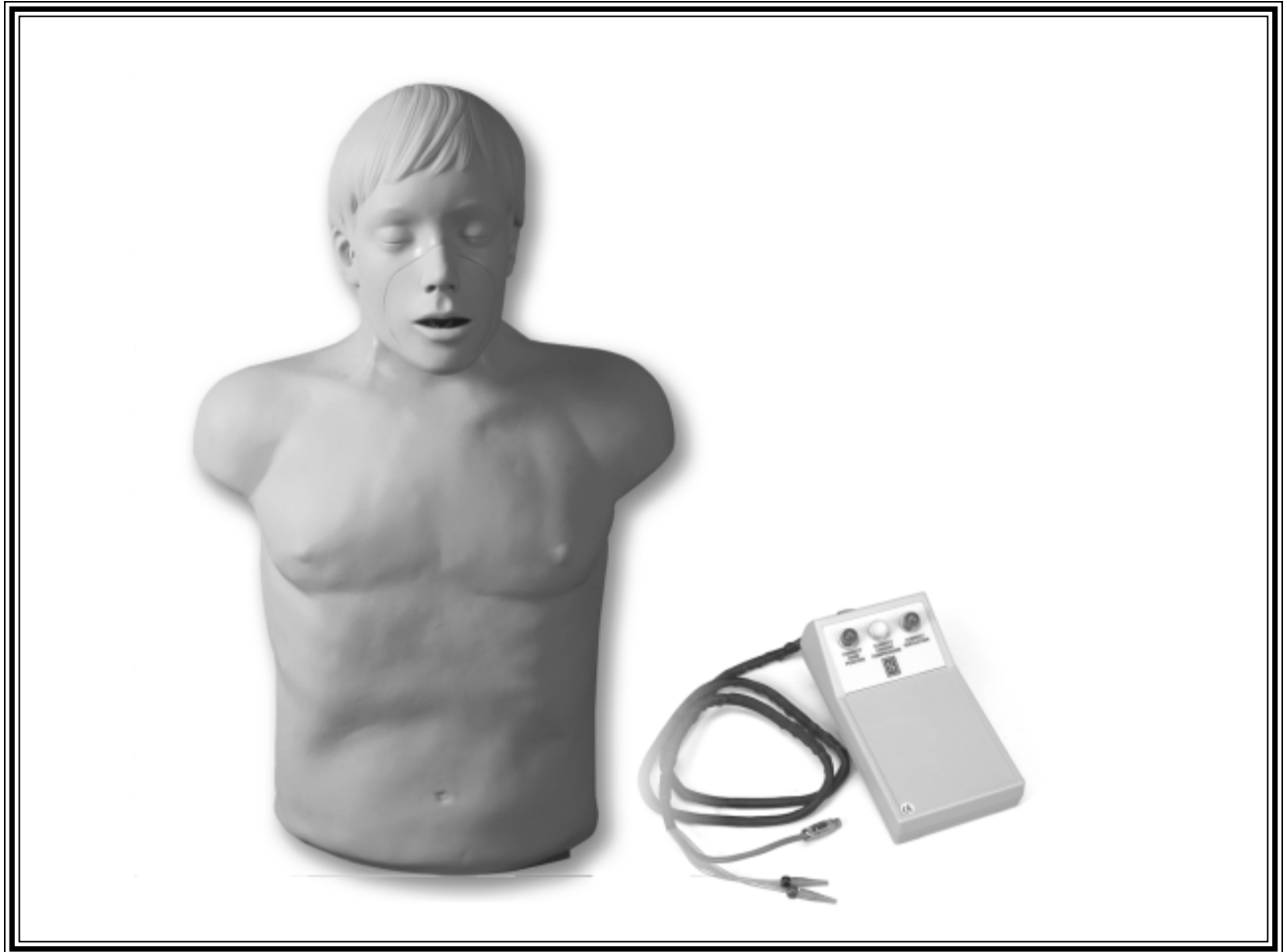

Electronic Brad Instructions for Use



 **Simulaids**

This instruction guide will familiarize you with Simulaids' Electronic BRAD CPR™ Manikin. Designed with your training needs in mind, the electronic BRAD CPR's realistic looks, accurate anatomical landmarks and individual-use mouth/nose pieces with disposable airway/lung system and non-rebreathing valve will continue to reinforce the manikin's value. The Manikin's sturdy vinyl covered torso means quick, wipe-clean maintenance.

Airway Removal:

1. Remove face insert and connector from inside face cavity.
2. With manikin face up, pull the overlay over the shoulders and fold down to expose chest plate and lung.
3. Tilt head back and pull one-way valve out of head.
4. Slip anchors from bottom of chest plate wings. Remove and discard lung.

Airway Installation:

1. Repeat step "2" above.
2. Attach exhaust manifold to opened bottom port on one-way valve.
3. Tilt head back and place one-way valve inside the head, inserting valve (fig.1) stem through hole in face. The lung inflation tube **MUST** be toward the bottom of the manikin to perform correctly.
4. Press connector on valve stem so flat end (fig. 2) of flange is toward mouthpiece.
5. Lay lung under chest plate and slip the lung anchors over the bottom of chest plate wings.
6. Replace chest skin overlay.



Fig. 1

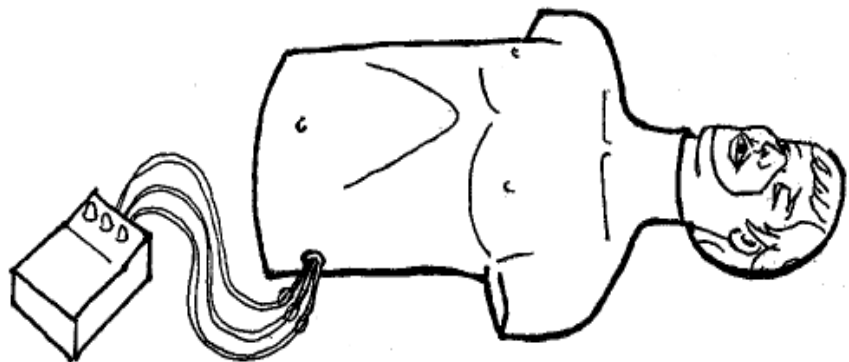


Fig.2

POWER SUPPLY / ELECTRONIC MODEL

The manikin console box contains 4 “AA” batteries (included). To replace batteries, slide open the panel from the underside of console box. Remove old batteries and replace with 4 new “AA” batteries in proper polarity

(+ or -) direction. Slide the panel back into place. Should lights fail to flash, either batteries are dead or have become disconnected. Check to make certain they are in correct position and that contact is being made. If bulb ever fails, return console to us for a free bulb replacement.



Cleaning & Disinfecting:

Manikin

Wash with warm, soapy water.

Rinse with clear water.

Mouth/ Nose pieces

Although Simulaids recommends disposing of Mouth/ Nose pieces at the end of class, they may be reused if desired using the following cleaning instruction

1. Clean in mild soapy water.
2. Rinse thoroughly.
3. Disinfect as per C.D.C. protocols.

Return Policy

Should it be necessary to return an item for any reason, contact our Customer Service Department to obtain an RGA Number. Please refer to your invoice number when phoning in your request for returning merchandise. Should you have any questions or wish further information on any product we manufacture call or write our Customer Service Department at: 800-431-4310

Warranty:

Simulaids warrants this product to be free from any defect in materials and/or workmanship for a period of three years from the date of purchase, as evidenced by the date of invoice when the product was shipped to the end user. This warranty expressly does not cover abuse, accidental or purposeful damage, or any form of modification to the product.

Simulaids reserves the right to either repair or replace affected parts or the entire unit, at their sole discretion, after investigating and reviewing the actual product and the damage. In most instances, a digital photo of the product in question showing the damage will help qualify a product for return to the factory. At no time will any product be accepted at the plant without proper return authorization issued by Simulaids. Freight and Shipping charges are the sole responsibility of the end user. No product will be received with shipping charges due.

Any product considered for warranty work must be identified by serial number and invoice number from the agency through whom the product was purchased. Without this information the product will not receive a return authorization number as required above.



Toll Free: (800)-431-4310 Fax: (845)-679-8996
www.simulaids.com