



**NO. 2710**  
**FLEXIBLE RESCUE RANDY**  
**TRAINER**

Thank you for purchasing a Simulaids product.


This vinyl manikin is made to bend at the waist and swivel at the head. The head swivels, the wrists swivel, the elbow, knee, shoulder and thigh joints move. The interior of the manikin is hollow around the waist so that it will bend by collapsing on itself. These features make it easier to conform the body to the positions necessary to maneuver the victim around the baffles in the wing tanks of airplanes. The manikin when exposed to low temperatures will become stiff. It will return to normal flexibility once it is returned to a higher temperature. It is suggested that you clothe the manikin in the same garments that are generally worn by your maintenance personnel. It will help to protect the manikin and give you greater realism as you train.

## **CARE AND MAINTENANCE**

Since the manikin takes the place of a live victim in your training program, it should be maintained with care and treated as you would any other valuable rescue training tool. To store the manikin lay it flat. Putting it on a back board and securing it will allow you to stand it in a corner. Full body storage bags are available, item 1373. Call your Simulaids distributor for pricing.

After each training exercise, or when needed, the manikin should be washed with hot, soapy water (do not immerse in water). The vinyl is porous and will hold stains. This includes ink. Do not mark on your manikin with ink; it will not come out. Do not store it with printed material as the ink will embed itself in the vinyl. The use of solvents or strong cleansers is not recommended. However, an "all -purpose" cleaner (i.e. 409 or Fantastic) may be used. Do not immerse the manikin in water or other fluids as the foam interior will hold the moisture and become infested with biological growth that you cannot remove.

# LEG ASSEMBLY DIAGRAM

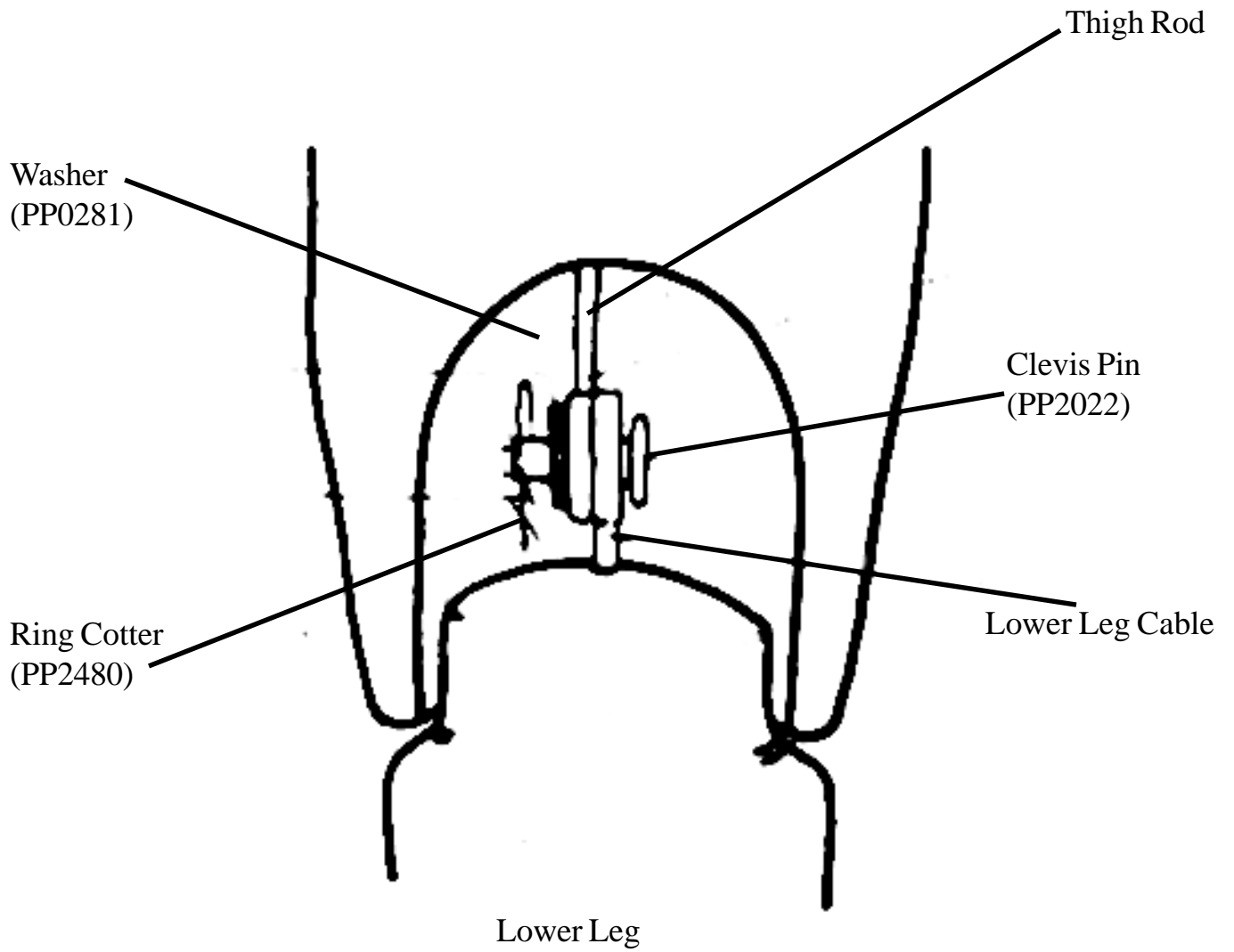

**RETURN GOODS POLICY** Should it be necessary to return an item for any reason, contact our Customer Service Department to obtain a RETURN AUTHORIZATION NUMBER. Merchandise cannot be accepted without prior return authorization. Please refer to your invoice number when phoning in your request for returning merchandise. If needed, a shipping carton and packing instructions will be sent to you to facilitate return of the item.

**WARRANTY** Simulaids warrants this product to be free from any defect in materials and/or workmanship for a period of three years from the date of purchase, as evidenced by the date of invoice when the product was shipped to the end user. This warranty expressly does not cover abuse, accidental or purposeful damage, or any form of modification to the product. Simulaids reserves the right to either repair or replace affected parts or the entire unit, at their sole discretion, after investigating and reviewing the actual product and the damage. In most instances, a digital photo of the product in question showing the damage will help qualify a product for return to the factory. At no time will any product be accepted at the plant without proper return authorization issued by Simulaids.

Freight and Shipping charges are the sole responsibility of the end user. No product will be received with shipping charges due. Any product considered for warranty work must be identified by serial number and invoice number from the agency through whom the product was purchased. Without this information the product will not receive a return authorization number as required above.



PO Box 1289 - Saugerties NY 12477

Toll Free (800) 431-4310

Fax (845) 679-8996

[www.simulaids.com](http://www.simulaids.com)

[info@simulaids.com](mailto:info@simulaids.com)