

# Installing STAT 2.0 on the iPAQ 110 Pocket PC

---

The iPAQ 110 runs Windows Mobile 6.0 and requires some additional steps to successfully install the STAT 2.0 Application. This document provides step by step instructions for successfully installing the STAT 2.0 software.

## Determine if STAT 2.0 is already installed

Before starting an installation it is necessary to determine if your Pocket PC already has the STAT 2.0 Application installed. Please perform the following steps and then proceed as instructed in Step 6.

1. Click the start menu on the Pocket PC
2. Tap on the “Programs” menu item to view all installed programs
3. Find the “File Explorer” program and tap on it
4. Next Tap on the “dropdown button” at the top left of the window. Then tap “My Device”. (see the below illustration)



5. Next Tap on the “Program Files” folder
6. Last look for a folder named “STAT\_2.\_0”.
  - a. If this folder **does exist** you need to perform a reinstallation. Click [here](#) to jump ahead to the “Re-Installing STAT 2.0” section of this document
  - b. If the “STAT\_2.\_0” folder **does not exist**, then please continue with the next section titled “Install from CD”

## Install from CD


---

There are 3 main steps for this process:


1. Install from CD
2. Turn off advanced USB functionality
3. Launch the STAT Application on the Pocket PC

## Step 1: Install from CD


1. Connect the iPAQ 110 Pocket PC to your Windows Computer
  - a. It is not necessary to create a partnership
2. Launch the STAT Installer CD on your Computer.
3. Proceed through the installation Sequence
  - a. Install STAT 2.0
  - b. Install the Manikin Data Editor
4. Next Start the installation of the STAT 2.0 for the Pocket PC application **\*\*Pay close attention to the Pocket PC screen at this time and watch for windows to appear on the Pocket PC**
5. The first warning is outlined in Red and has a Yes button. Tap the Yes button to dismiss the warning.

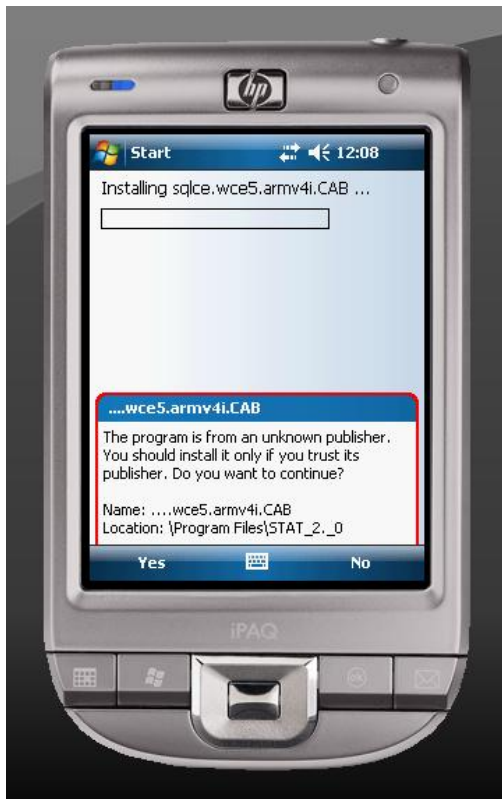

6. When the installation is complete you will see the following screen. Tap OK to dismiss it.

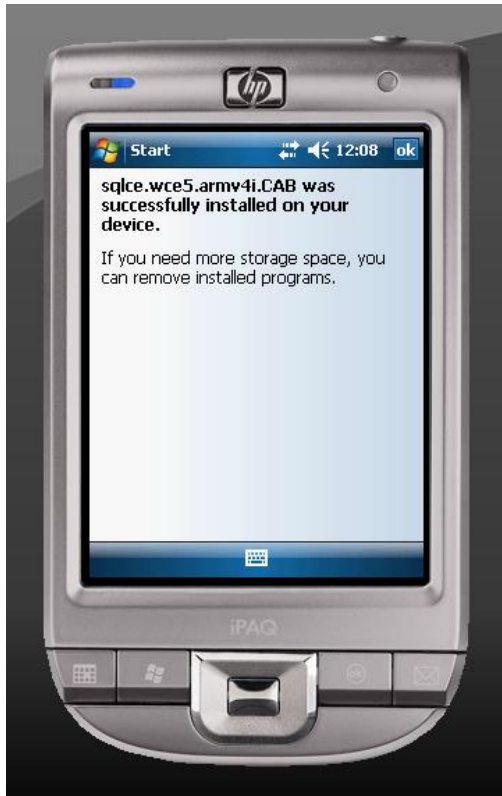

7. Next the following warning should appear. Again, tap Yes to dismiss this warning.

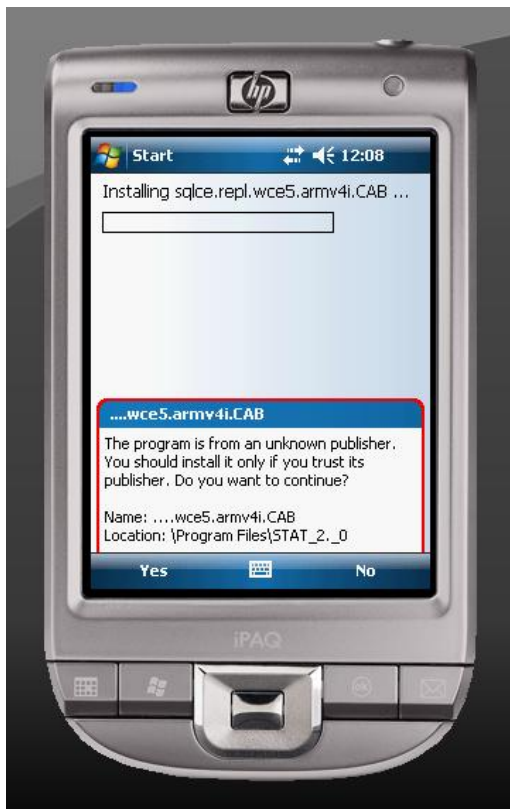

8. When the installation is complete tap “OK” on the below screen




9. The next warning should appear when the installation is complete and the sync operation begins with the desktop. It will look like the below screen and is once again outlined in red. Please tap “yes” to allow the sync to continue.




10. Wait for the sync process to finish. When the sync (and installation) are complete disconnect the Pocket PC from the windows computer.

## Step 2: Turn off advanced USB functionality

1. After installation it is necessary to turn off the advanced USB functionality
2. Tap the Start Button to open the start menu
3. Tap the settings menu item.
4. Then tap the connections tab (circled in Orange below)

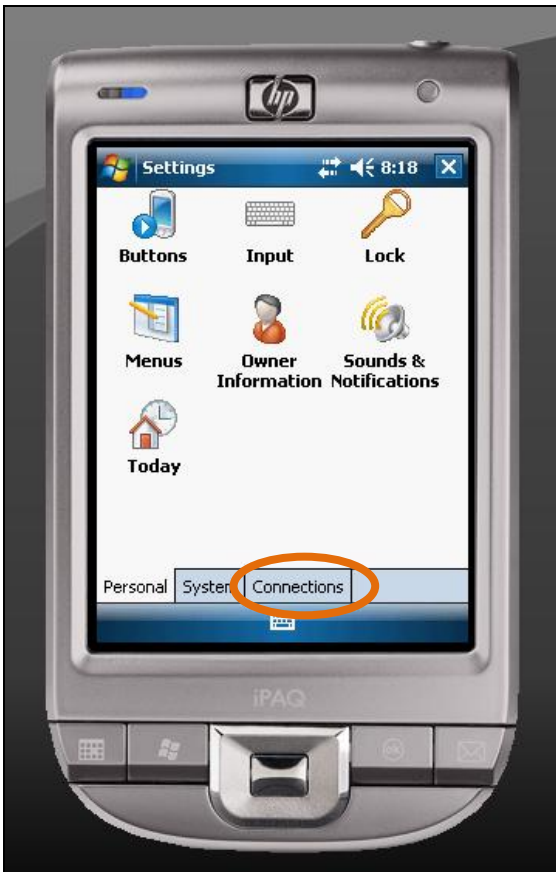

5. Tap the USB to PC icon.
6. Then remove the checkmark from the “enable advanced network functionality” checkbox

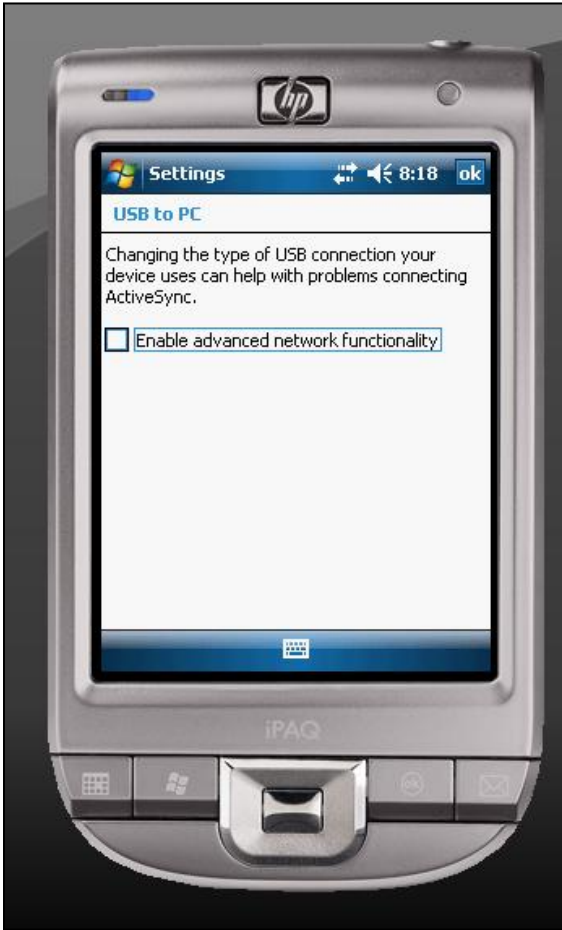


7. Tap OK to save changes

### Step 3: Launch the STAT Application on the Pocket PC

1. Now launch the STAT application on the Pocket PC. You will see multiple warnings when the application is launching. You will need to tap Yes on each of the below warnings (see next page if necessary).
  - a. NOTE: These will only appear the first time you run the application.
  - b. NOTE: Please be patient for the "CommSim.dll" warning, it takes longer to dismiss than the rest of the warnings
2. There are 5 warning messages, in the following order:



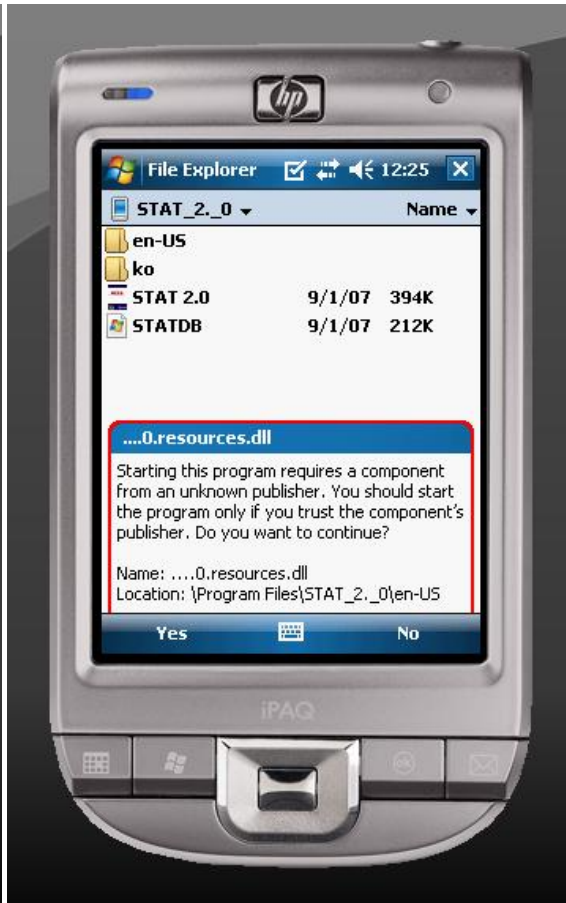


# Re-Install from CD

---


If your Pocket PC currently has STAT 2.0 preinstalled there is only one step required for re-installation.

## Step 1: Install from CD


1. Connect the iPAQ 110 Pocket PC to your Windows Computer
  - a. It is not necessary to create a partnership
2. Launch the STAT Installer CD on your Computer.
3. Proceed through the installation Sequence
  - a. Install STAT 2.0
  - b. Install the Manikin Data Editor
4. Next Start the installation of the STAT 2.0 for the Pocket PC application **\*\*Pay close attention to the Pocket PC screen at this time and watch for windows to appear on the Pocket PC**
5. The first warning is a small window with an OK and Cancel button. Tap OK.

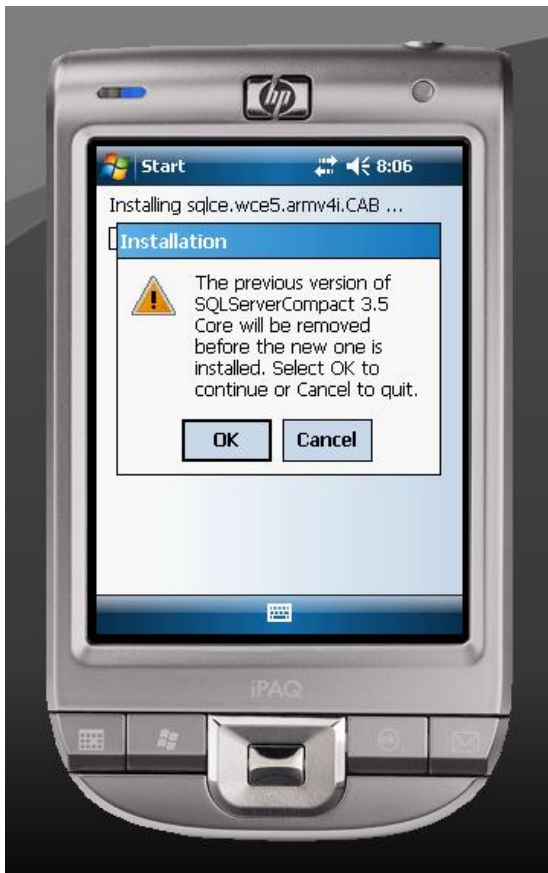

6. Wait for the installation to finish, then Tap OK when the below screen is displayed

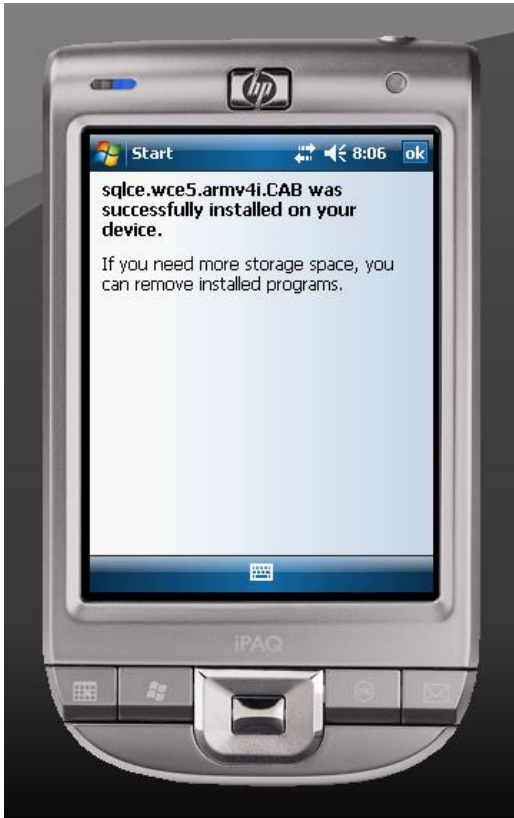

7. Now wait for the below screen and Tap OK when you see the small window

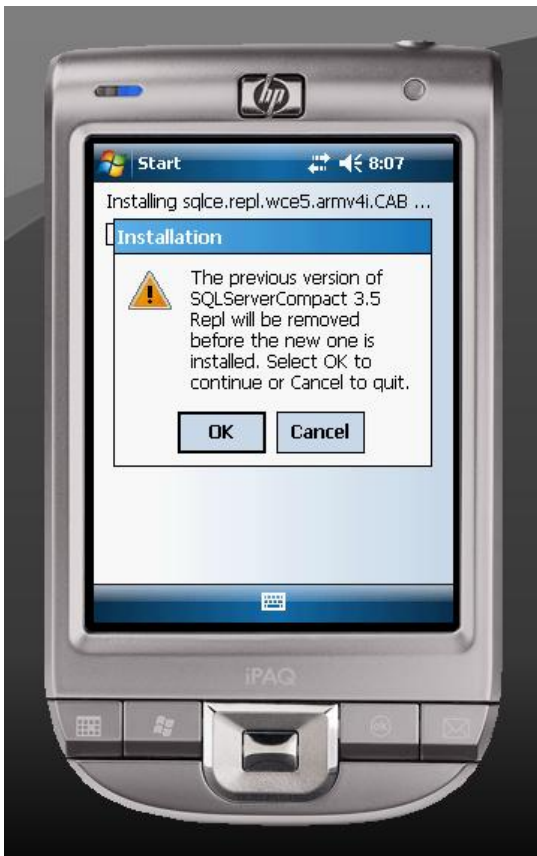

8. Then tap OK when the below screen is displayed



9. Installation is complete. Disconnect the USB cable and begin using STAT 2.0.