



---

**Instruction Guide**  
**for BRAD JR CPR™**



## **Thank you for buying an American made product.**

This instruction guide will familiarize you with Simulaids' BRAD JR CPR™ Manikin. Designed with your training needs in mind, BRAD JR CPR's realistic looks, accurate anatomical landmarks and individual-use mouth/nose pieces with disposable airway/lung system and non-rebreathing valve will continue to reinforce the manikin's value. The Manikin's sturdy vinyl covered torso means quick, wipe-clean maintenance.

### **Airway Removal:**

1. Remove mouth/nose piece and connector from inside face cavity.
2. With manikin face up, pull the overlay over the shoulders and fold down to expose chest plate and lung.
3. Tilt head back and pull one-way valve out of head.
4. Slip anchors from bottom of chest plate wings. Remove and discard lung.

### **Airway Installation:**

1. Press red exhaust manifold cap onto the opened port of the one-way valve next to the port of the lung attachment.
2. Tilt head back, insert valve stem (fig. 1) through hole in face. The lung inflation tube MUST be toward the bottom of the manikin to perform correctly. Press connector onto valve stem (fig. 2) DO NOT PRESS straight on. Angle and apply twisting motion while pressing forward.
3. Lay lung under chest plate and slip the lung anchors over the bottom of chest plate wings.
4. Pull chest skin up and over shoulders.
5. Replace mouth/nose piece.

Fig:1

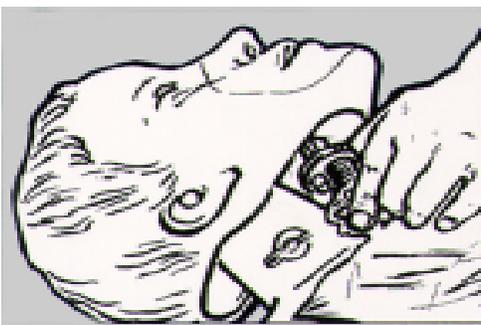


Fig:2



## **Cleaning & Disinfecting:**

### **Manikin**

Wash with warm, soapy water.

### **Mouth/ Nose pieces**

Although Simulaids recommends disposing of Mouth/ Nose pieces at the end of class, they may be reused if desired using the following cleaning instructions.

1. Clean in mild soapy water.
2. Rinse thoroughly.
3. Follow local protocol for disinfection.
4. Rinse thoroughly.

**DO NOT USE BLEACH FULL STRENGTH**

<b>Part #</b>	<b>Description</b>
2271	Brad Jr with carry bag
2280	Airway/ lung system (24/ pk.)
2223	Mouth/ Nose Piece (10/pk.)
2526	Carry bag with kneeling pads

**Return Policy:** Should it be necessary to return an item for any reason, contact our Customer Service Department to obtain an RGA Number. Please refer to your invoice number when phoning in your request for returning merchandise. Should you have any questions or wish further information on any product we manufacture call or write our Customer Service Department.

**Warranty:** Simulaids warrants this product to be free from any defect in materials and/ or workmanship for a period of three years from the date of purchase, as evidenced by the date of invoice when the product was shipped to the end user. This warranty expressly does not cover abuse, accidental or purposeful damage, or any form of modification to the product. Simulaids reserves the right to either repair or replace affected parts or the entire unit, at their sole discretion, after investigating and reviewing the actual product and the damage. In most instances, a digital photo of the product in question showing the damage will help qualify a product for return to the factory. At no time will any product be accepted at the plant without proper return authorization issued by Simulaids.

Freight and Shipping charges are the sole responsibility of the end user. No product will be received with shipping charges due. Any product considered for warranty work must be identified by serial number and invoice number from the agency through whom the product was purchased. Without this information the product will not receive a return authorization number as required above.



 **Simulaids**



Manufacturers of Training Manikins, Casualty  
Simulation Kits, Medical Training Devices  
PO Box 1289, 16 Simulaids Ave,  
Saugerties NY 12477  
(845)-679-2475  
Toll Free: (800)-431-4310 Fax: (845)-679-8996  
[www.simulaids.com](http://www.simulaids.com)