

# Simulaids

## CasPeR



No. 5000

This small animal trainer incorporates all the necessary features for teaching basic dog CPR to pet owners. The anatomy features a full body, molded in the appropriate position in which to give CPR, nostrils, retractable tongue, moveable jaw, sanitary replaceable airway, foam-filled body for chest compression resistance, and a femoral pulse.

## CasPeR Airway Installation



2

Pull out foam block.



3

Pull out used airway.



4

Fold new airway in half 4 times.



5

Pinch nostrils and insert closed end of airway. Push airway in about half way.



6

Reach into chest cavity and pull airway out of the airway tube.



7

Unfold the airway and spread it out in the bottom of the chest cavity. Replace the foam block and the back plate.



8

Spread the airway bag over the snout. Inserting a finger into the nasal passage will ensure that the airway is open.

Install the pulse bulb assembly tubing into the port located on CasPeR's hindquarter. A simulated femoral pulse can be felt on the inside of the upper leg when the bulb is rhythmically squeezed.

**CasPeR Lung 100 Pack No. 5001**

**CARE AND MAINTENANCE:**

Using a solution of mild soap and warm water, clean with a damp cloth and allow to thoroughly dry before storing.

**CAUTION:** Do not allow foamed portion of manikin to become wet. If the foamed portion of the manikin becomes wet, allow it to dry thoroughly before storing.

**RETURN GOODS POLICY:**

Should it be necessary to return an item for any reason, contact our Customer Service Department to obtain a RETURN AUTHORIZATION NUMBER. Merchandise cannot be accepted without prior return authorization. Please refer to your invoice number when phoning in your request for returning merchandise. Should you have any questions or wish further information on any product we manufacture or distribute, contact our Customer Service Department.

**WARRANTY:**

Simulaids warrants this product to be free from any defect in materials and/or workmanship for a period of three years from the date of purchase, as evidenced by the date of invoice when the product was shipped to the end user. This warranty expressly does not cover abuse, accidental or purposeful damage, or any form of modification to the product. Simulaids reserves the right to either repair or replace affected parts or the entire unit, at their sole discretion, after investigating and reviewing the actual product and the damage. In most instances, a digital photo of the product in question showing the damage will help qualify a product for return to the factory. At no time will any product be accepted at the plant without proper return authorization issued by Simulaids.

Freight and Shipping charges are the sole responsibility of the end user. No product will be received with shipping charges due. Any product considered for warranty work must be identified by serial number and invoice number from the agency through whom the product was purchased. Without this information the product will not receive a return authorization number as required above.

PO Box 1289 - 16 Simulaids Drive  
Saugerties NY 12477  
Tel. 845-679-2475 - Fax 845-679-8996  
Toll Free 800-431-4310  
[www.simulaids.com](http://www.simulaids.com)  
email - info@simulaids.com



 **Simulaids**



[www.simulaids.com](http://www.simulaids.com)