
Rescue Jennifer & Combat Challenge Kid

Instructions for Use



 **Simulaids**

RESCUE JENNIFER MANIKIN

This manual will familiarize the user with the function, care and maintenance of the RESCUE JENNIFER rescue training manikin. As with all SIMULAIDS' products, RESCUE JENNIFER is made to exacting standards and will provide trouble-free service when the few guidelines and simple steps of care and maintenance are followed.

FEATURES

RESCUE JENNIFER is the size of a 7- to 12 year old adolescent. Articulated at all major joints with rugged construction engineered for years of durability and usefulness. Especially helpful for preparing personnel for child rescue where highly charged emotional factors come into play.

CARE AND MAINTENANCE

RESCUE JENNIFER should be maintained with care and treated as you would any other valuable rescue training tool. It will withstand rugged use. After each training exercise, or when needed, RESCUE JENNIFER should be washed with hot, soapy water. The use of solvents or strong cleansers is not recommended. However, an "all-purpose" cleaner (i.e., 409 or Fantasic) may be used. RESCUE JENNIFER may be used in low temperatures, but if so, you can expect the vinyl body to become stiff. It will return to normal flexibility once it is returned to a higher temperature.

- #1355 Rescue Jennifer 4 ft., 16 lbs.
- #1355SPEC Rescue Jennifer 4 ft., 38 lbs
- #1460 Combat Challenge Kid 13lbs.
- #3100 Carry Bag

WARRANTY: Simulaids warrants this product to be free from any defect in materials and/or workmanship for a period of three years from the date of purchase, as evidenced by the date of invoice when the product was shipped to the end user. This warranty expressly does not cover abuse, accidental or purposeful damage, or any form of modification to the product. Simulaids reserves the right to either repair or replace affected parts or the entire unit, at their sole discretion, after investigating and reviewing the actual product and the damage. In most instances, a digital photo of the product in question showing the damage will help qualify a product for return to the factory. At no time will any product be accepted at the plant without proper return authorization issued by Simulaids.

Freight and Shipping charges are the sole responsibility of the end user. No product will be received with shipping charges due. Any product considered for warranty work must be identified by serial number and invoice number from the agency through whom the product was purchased. Without this information the product will not receive a return authorization number as required.

RETURN POLICY: Should it be necessary to return an item for credit for any reason, contact our Customer Service Department to obtain an RGA Number. Please refer to your invoice number when phoning in your request for returning merchandise. Should you have any questions or wish further information on any product we manufacture call or write our Customer Service Department.



 **Simulaids**



**Manufacturers of Training Manikins, Casualty Simulation Kits,
Medical Training Devices**

PO Box 1289 - 16 Simulaids Dr - Saugerties NY 12477

(845)-679-2475

Toll Free: (800)-431-4310 Fax: (845)-679-8996

www.simulaids.com