




Assembly & Operation Instructions

KIM, KEVIN & KYLE AIRWAY INSTALLATION

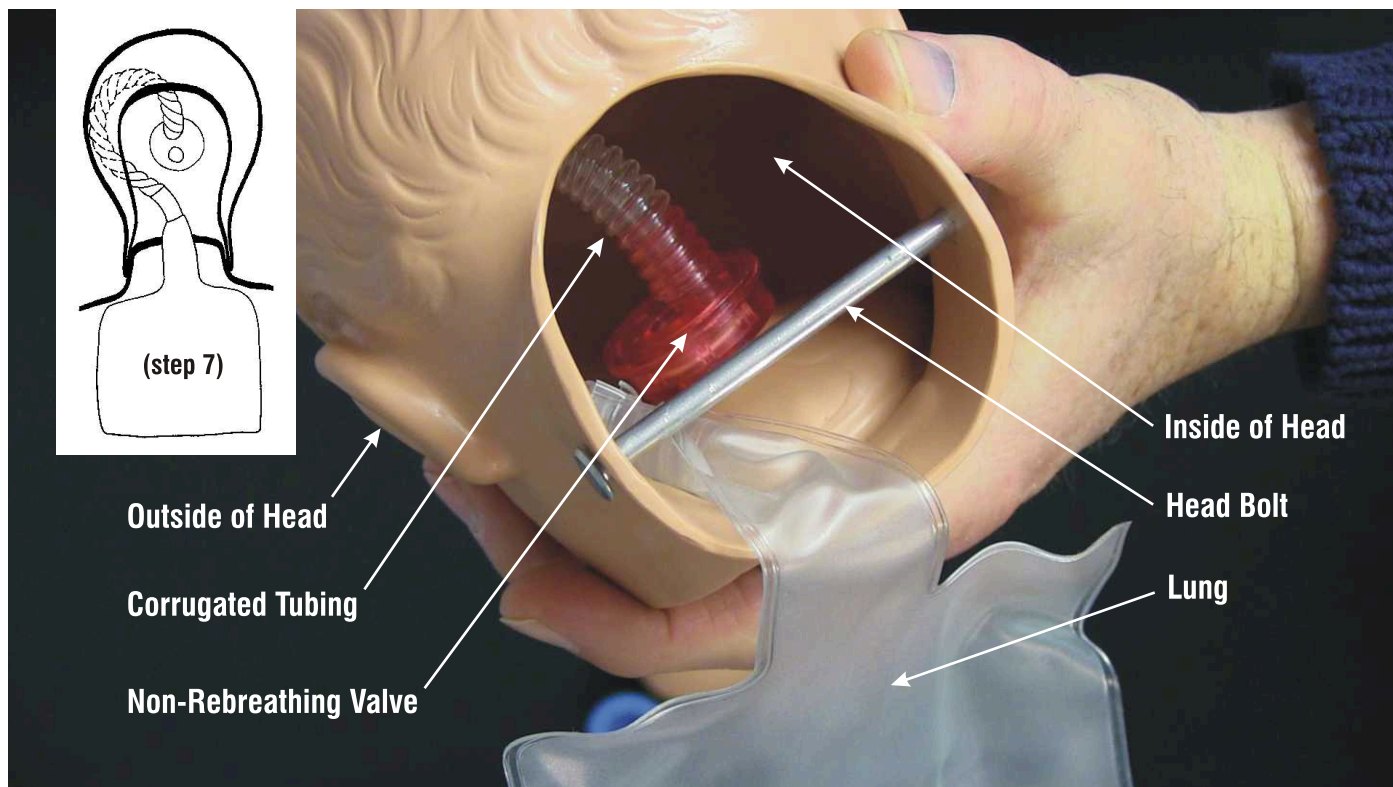

Airway Installation

- Steps:**
1. Unsnap the chest overlay and remove it from the manikin.
 2. Pull back the retaining pin located at the base of the neck.
 3. Lift the head off the body and remove the mouth/face piece.
 4. Insert the non-rebreathing valve through the back of the face cavity.
 5. Press the flanged valve (a slight twisting motion assists with placement) onto the non-rebreathing valve stem.
 6. Replace the mouth/face piece.
 7. Loop tubing inside of the head for both Kim and Kevin (note the position in "step 7" illustration).
 8. Position the airway/lung over the front of the head bolt.
 9. Direct the elbow joint on the Kyle airway so it points straight down.
 10. Pull back the retaining pin located at the base of the neck.
 11. Attach the head to the body.
 12. With head slightly tilted back, lay the lung flat against the chest.
 13. Snap the overlay in place.